

## HANDBOOK

### ANTIX GOOGLES



X – ray protective glasses are researched and produced specially for X – ray protection, which should be used kept carefully due to they are made of lead glass lenses in special process.

- Lead glasses should not be hanged under neck with the rope due to it is heavy.
- After using please place the lead glasses into box immediately to avoid its damaged.

The lead glassed will not be guarantee if damaged by the actions above.

- The lenses of lead glasses are not of safe lenses.

The lead glasses should not be used if the circumstances might make the lenses fall down.

- The lenses are not anti – broken glass. If there is crack in lenses, lead glasses should not be used any more in order to avoid injured. In this case, it needs to be changed we new lenses.