



Via Michelangelo Buonarroti, 52/B - 20093 Cologno M.se (Mi)

## **TANK TYPE ALFA 2 MANUAL MODEL INDUSTRY**



## **INSTRUCTION AND INSTALLATION HANDBOOK**



REV.01/2015


## 1.0 INTRODUCTION

Construction in acid-proof pvc in order to grant longer quality, reliability and guarantee.

The tank is complete of 1 extractable tank lt. 10 for developing.

One extractable tank lt. 10 for fixing.

The intermediate and final washing area has a capacity of 80 lt.

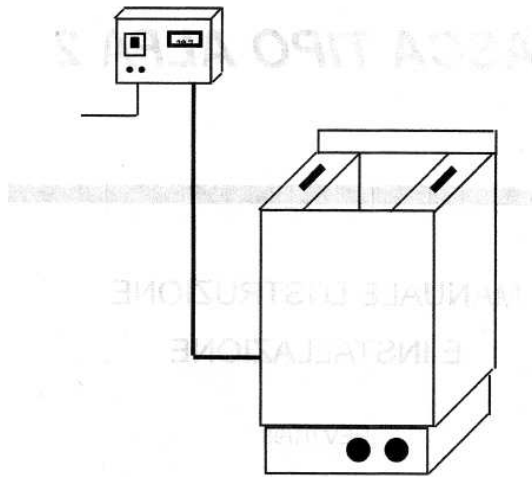

## 2.0 INSTALLATION

Before proceeding with installation, PLS control that the material delivered is complete in all parts.

- **SUPPORT BASIS**

In order to avoid unwanted overturning, always verify that the tank is on a plan.

**THE CABLES WHICH CONNECT THE TANK TO THE ELCTRIC PANEL CAN'T ABSOLUTELY BE EXTENDED OR MANUMITTED.**

## 3.0 ACTIVATION

- Insert the plug into the outlet of the tank, lift the lever of this general framework in the "ON" position and the unit is running.

#### **4.0 MAINTENANCE**

The tank type ALFA 2 model industry doesn't need a special maintenance.

We suggest to completely discharge the tank to avoid the lodging of limestone and other incrustations on the electrical resistor and also on other parts of the tank.

#### **5.0 INNER PART REPLACEMENT**

If the alimentation cable is damaged, it has to be replaced from suppliers or from an authorized assistance centre.

#### **6.0 GUARANTEE**

The repair and replacement of faulty parts is under guarantee. Guarantee doesn't include indirect damage and every kind of consumer material is excluded.

Guarantee is also not valid in case of failure in observing rules and instructions included in this handbook.

**ATTENTION: THIS HANDBOOK MUST BE KEPT DURING ALL THE LIFE OF TANK ALFA 2 MODEL INDUSTRY AND MUST BE ALWAYS AT DISPOSAL OF OPERATORS AND MAINTENANCE STAFF.**

**CABLAS s.r.l. company decline every kind of responsibility in case of omission or tempering of the device.**