



Via Michelangelo Buonarroti, 52/B - 20093 Cologno M.se (Mi)

TANK TYPE ALFA 2



INSTRUCTION AND INSTALLATION HANDBOOK



REV.01/2015

1.0 INTRODUCTION


Construction in acid-proof pvc in order to grant longer quality, reliability and guarantee.

The tank has a developing area with thermoregulated and armored resistors.

The tank is complete of 1 extractable tank lt. 10 for developing.

One extractable tank lt. 10 for fixing.

The intermediate and final washing area has a capacity of 80 lt.

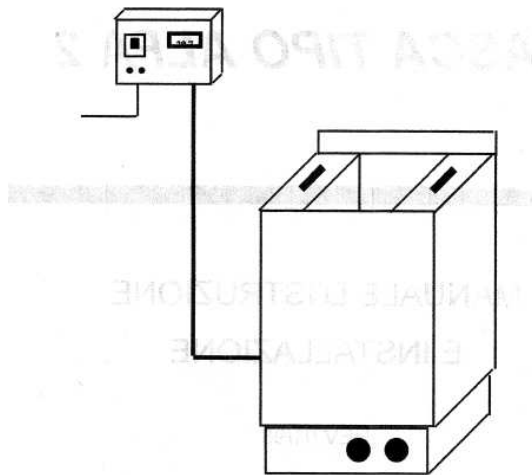

N.B. FIX THE ELECTRICAL PANEL AT A HIGH ACCESSIBLE TO THE OPERATOR

ATTENTION: BEFORE CONNECTING THE PLUG, ASSURE THAT THE TANK CONTAINS WATER, OTHERWISE YOU COULD DAMAGE THE RESISTOR.

2.0 INSTALLATION

Before proceeding with installation, PLS control that the material delivered is complete in all parts.

- **SUPPORT BASIS**

In order to avoid unwonted overturning, always verify that the tank is on a plan.

- **ELECTRICAL CONNECTION**

The tank is supplied with alimentation cable with plug. The commitment has only to arrange a plate near the device with capacity equal or over 6A with **HEARTH**.

THE CABLES WHICH CONNECT THE TANK TO THE ELCTRIC PANEL CAN'T ABSOLUTELY BE EXTENDED OR MANUMITTED.

3.0 ACTIVATION

Using the tank , please be advised that the developing and fixing acid are harmful whether are in touch with some body parts, **HANDLE WITH CARE**. They are also considered dangerous rubbish and must be eliminated according to the Government's rules.

4.0 REGULATION

There is a thermostat in the device, You can regulate the liquid temperature at the optimal value suggested by the supplier.

5.0 PARAMETERS PROGRAMMING

SET: pushing and leaving it again You will obtain the visualization of temperature value fixed. If You keep pushing SET button and work with red and light blue arrows, You can raise or reduce the value You fixed.

dT: differential. intervention differential. To regulate the differential, push the proper button and work with arrow keys, so You can raise or reduce the value.

6.0 MAINTENANCE

The tank type ALFA 2 thermoregulated doesn't need a special maintenance.

We suggest to completely discharge the tank to avoid the lodging of limestone and other incrustations on the electrical resistor and also on other parts of the tank.

7.0 INNER PART REPLACEMENT

If the alimentation cable is damaged, it has to be replaced from suppliers or from an authorized assistance centre.

8.0 GUARANTEE

The repair and replacement of faulty parts is under guarantee. Guarantee doesn't include indirect damage and every kind of consumer material is excluded.

Guarantee is also not valid in case of failure in observing rules and instructions included in this handbook.

ATTENTION: THIS HANDBOOK MUST BE KEPT DURING ALL THE LIFE OF TANK ALFA 2 THERMOREGULATED AND MUST BE ALWAYS AT DISPOSAL OF OPERATORS AND MAINTENANCE STAFF.

CABLAS s.r.l. company decline every kind of responsibility in case of omission or tempering of the device.