



Via Michelangelo Buonarroti, 52/B - 20093 Cologno M.se (Mi)

TANKS DEVELOPMENT MANUAL MODEL INDUSTRY



INSTRUCTION AND INSTALLATION HANDBOOK



REV.01/2015

1.0 INTRODUCTION


Construction in acid-proof pvc in order to grant longer quality, reliability and guarantee.

The tank is complete of 1 extractable tank lt. 20 for developing.

One extractable tank lt. 20 for fixing.

The intermediate washing area has a capacity of 16 lt.

The final washing area has a capacity of 16 lt.

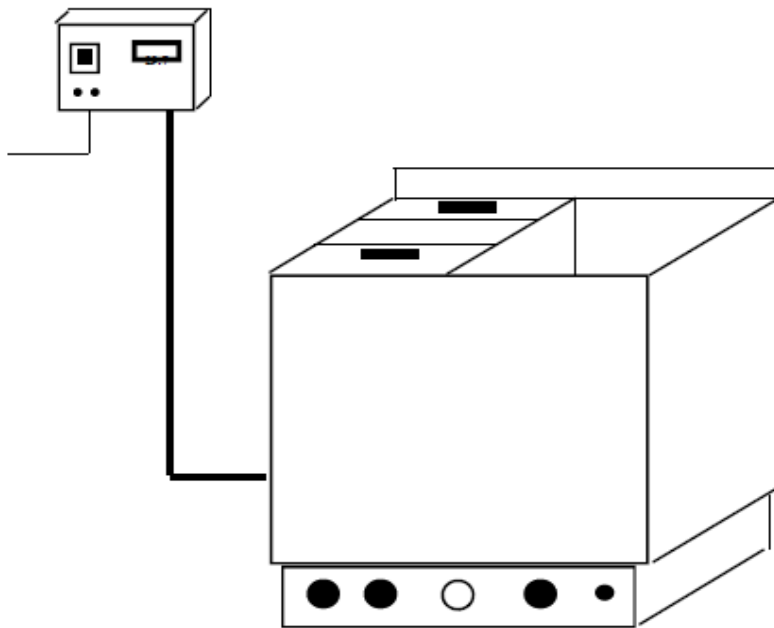

2.0 INSTALLATION

Before proceeding with installation, PLS control that the material delivered is complete in all parts.

- **SUPPORT BASIS**

In order to avoid unwanted overturning, always verify that the tank is on a plan.

THE CABLES WHICH CONNECT THE TANK TO THE ELCTRIC PANEL CAN'T ABSOLUTELY BE EXTENDED OR MANUMITTED.

3.0 ACTIVATION

Insert the plug into the electrical plate, lift up the lever that You can find in the general panel in "ON" position and the device will operate.

4.0 MAINTENANCE

The tank type DELTA 1 model industry doesn't need a special maintenance.

We suggest to completely discharge the tank to avoid the lodging of limestone and other incrustations.

5.0 INNER PART REPLACEMENT

If the alimentation cable is damaged, it has to be replaced from suppliers or from an authorized assistance centre.

6.0 GUARANTEE

The repair and replacement of faulty parts is under guarantee. Guarantee doesn't include indirect damage and every kind of consumer material is excluded.

Guarantee is also not valid in case of failure in observing rules and instructions included in this handbook.

ATTENTION: THIS HANDBOOK MUST BE KEPT DURING ALL THE LIFE OF TANK DELTA 1 MODEL INDUSTRY AND MUST BE ALWAYS AT DISPOSAL OF OPERATORS AND MAINTENANCE STAFF

CABLAS s.r.l. company decline every kind of responsibility in case of omission or tempering of the device.